

Creating a high-capacity overnight drainage solution


For indwelling or male external catheters

Conveen®

The Conveen® Night bag can be connected to a Coloplast Conveen® leg bag for extra drainage capacity during the night.

Use the Conveen bag hanger to suspend the night bag.
Fit the bag securely on the hanger by inserting the teeth of the hanger in the slits at the top of the bag.
Hook the bag hanger onto the bed.

Tip: To ensure optimal drainage, it is important that the night bag is positioned lower than yourself, when you are lying in bed.

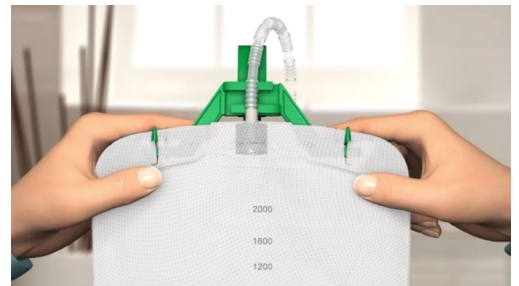

If the connector has a protective cap, remove it.
Attach the night bag connector securely to the leg bag outlet tap.
Open the leg bag outlet tap.



The urinary collecting system is now functional.

Tip: When in bed, loosen the straps of the leg bag for greater comfort.

To disconnect:

Close the leg bag outlet tap
Disconnect the night bag by gently twisting the connector out of the leg bag outlet tap.
Avoid touching the open end of the leg bag outlet tap.

Tip: It is recommended to empty the night bag when it is a little over half full. This will prevent air pressure build-up inside the bag which could cause problems with drainage.



For instructional videos visit:
www.coloplast.ca/user-guides

To empty the night bag:

Open the outlet tap by gently pushing the tap arm downwards until you feel resistance.

Then press the tap arm sideways into the barrel to allow the urine to flow.



Remember to close the outlet tap after emptying the bag. Return the tap arm to its upright position by pushing it upwards.




Both the bag and the packaging can be disposed of in the household waste.

Tip: Change the bag according to recommendations by your health care professional.



Product options

Conveen Urine bags		Volume	Tube length	Product code	Units
	Conveen® Security+ Leg Bag <ul style="list-style-type: none">• Designed for extra security• Clamp urine outlet• Anti-kink tube for free flow• Soft backing for increased comfort• PVC and phthalate free• Non sterile	750 mL	20"	5167	10
	Conveen® Night Bag <ul style="list-style-type: none">• Large capacity bedside bag• Anti-kink tube for free flow• Two step outlet• PVC and phthalate free• Sterile	2000 mL	55"	21356	10

Accessories		Length cm/in	Product code	Units
	Conveen® Bag Hanger	18 cm / 7"	5070	1
	Conveen® Leg Bag Straps <ul style="list-style-type: none">• Sold in pairs	63 cm / 25"	5050	10
	Conveen® Catheter retaining strap	90 cm / 35"	383004	2