

Triad[®]

Hydrophillic wound dressing



A **unique** approach

to wound management

Impact of *difficult-to-dress* wounds

On wet or irregular surfaces, wound dressings may not always stay in place. Frequent dressing replacements could lead to:

Avoidable costs related to using extra dressings

Increased workload for nurses replacing dressings

Delayed wound healing impacting the total cost of care

Did you know?

57%

of wound care nurses see difficult-to-dress wounds every day¹

How do difficult-to-dress wounds impact your facility?



How *Triad*[®] works



Wounds on *irregular surfaces*

Triad is designed to be applied directly from the tube onto the wound without the need for a secondary dressing.

Irregular surfaces could include the gluteal cleft, coccyx, perineum, buttocks, groin, face, hands, feet and other areas.

Triad is easy to apply, and can be used on the wound or peri-wound skin.

Did you know?

64%

of pressure injuries occur on the sacrum or buttocks²



Broken skin in the presence of incontinence

Triad is a sterile coating that can be used on broken skin.

CMC allows Triad to adhere to wet skin, keeping the wound covered and protected from incontinence.

Triad contains dimethicone to moisturize the skin and petrolatum and zinc oxide to reduce skin irritation.

Did you know?

26%

of patients in acute care have urinary continence³



Maceration of peri-wound skin

Triad is easy to use on larger surfaces and can be used on the wound and peri-wound skin.

CMC allows Triad to adhere to macerated skin, keeping it covered and protected from wound exudate.

Triad contains dimethicone to moisturize the skin and petrolatum and zinc oxide to reduce skin irritation.

In one case, Triad resolved extensive peri-wound maceration within 48 hours.⁵



Day 1

Application

Day 2

Did you know?

42%

of chronic wounds
are macerated⁴



Necrotic tissue such as slough or eschar

Triad is hydrophilic, which means that natural moisture spreads evenly across the wound surface, maximizing contact and creating a moist environment.

A moist wound environment facilitates autolytic debridement, where the body's own enzymes break down necrotic tissue.

CMC allows Triad to absorb low to moderate levels of wound exudate or broken-down tissue.

In one case, Triad debrided 100% thick black eschar within one week.⁶



Day 1

Day 12

Day 21

Did you know?

Necrosis

is a barrier to
wound healing

Triad®

Hydrophilic wound dressing

A unique approach

Triad® Hydrophilic Wound Dressing is a sterile coating that offers a unique approach to wound management.

What do you currently use for difficult-to-dress wounds?



Apply anywhere

Triad is easy to apply, directly from the tube onto the wound or peri-wound skin, anywhere on the body.



Adheres to wet skin

Triad adheres to wet or dry skin and consistently stays in place, even in the presence of incontinence or maceration.



Keeps the wound covered

Triad creates a moist wound environment that facilitates healing and autolytic debridement.



“With Triad, we’ve saved significantly and had better outcomes. We don’t need a nurse to change the dressing every day and that’s had a big impact on saving time and material.”

H KIRKLAND-KHYN,
Director of Wound Care, University of California, Davis, Medical Center

Triad® instructions for use video:

Scan or text below to view the video:

Scan QR code using your smart phone camera.
Launch video by tapping the link that appears on your screen (not compatible with all smart phones).



Or, text **94253** and enter keyword **"triad"** to view video.

Ordering Information

Triad® Hydrophilic Wound Dressing

Code	Size	Units
1964	2.5 oz. (71 g)	12
1967	6 oz. (170 g)	12



Sea-Clens® Wound Cleanser

A pH balanced wound cleanser that can be used to remove Triad without scrubbing.

Code	Size	Units
1063	6 fl. oz. (178 mL)	12
1067	12 fl. oz. (355 mL)	12



Add a new tool to your wound management toolbox, TryTriad.ca

References

1. Online questionnaire conducted by Coloplast on 7/30/2019 with 395 healthcare professionals.
2. Bauer et al., Pressure Ulcers in the United States' Inpatient Population, Ostomy Wound Management, 2016.
3. Condon et al., Urinary and Faecal Incontinence: Point Prevalence and Predictors, IJERPH, 2019.
4. Digital survey conducted by Coloplast during Spring 2018 with 2000 healthcare professionals.
5. Mullen et al., Management of wounds with various etiologies using a hydrophilic wound dressing, WOCN, 2013.
6. Boyle et al., Using a hydrophilic wound dressing for autolytic debridement, SAWC, 2010.

Triad is only sterile on first application.

[Ostomy Care](#) / [Continence Care](#) / [Wound & Skin Care](#) / [Interventional Urology](#)

Coloplast United States
Minneapolis, MN 55411 / 1-800-533-0464
www.coloplast.us

Coloplast Canada
Oakville, ON L6H 5S9 / 1-877-820-7005
www.coloplast.ca

The Coloplast logo is a registered trademark of Coloplast A/S.
© 2020 Coloplast Corp. All rights reserved.

